


## POLIGRIP POWER HOLD + SEAL DENTURE ADHESIVE

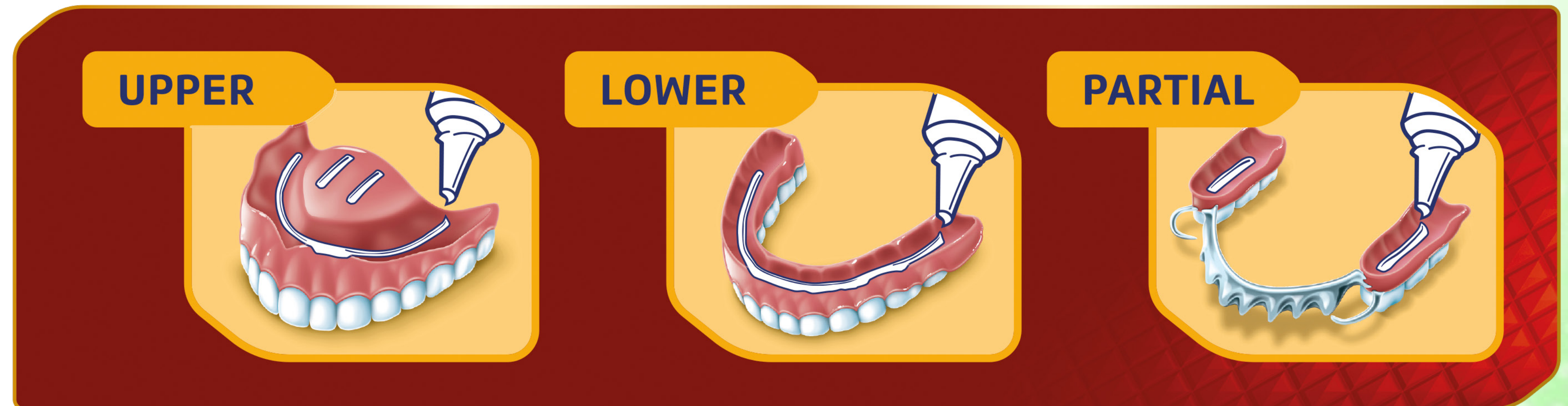

The precision nozzle enables exactly the right amount of product to be applied around the edge of the denture forming an all-around seal

### APPLYING:

1. Clean and dry your dentures
2. Apply Poligrip as shown in the diagram, not too close to the denture edges.
  - a. **Upper denture:** one long continuous strip with two shorter strips in the middle
  - b. **Lower denture:** one long continuous strip
  - c. **Partial denture:** two short strips
3. Rinse mouth before inserting dentures
4. Press dentures into place, hold firmly and bite down for a few seconds to secure hold

### REMOVING:

1. Swish mouth with water
2. Slowly remove denture using a rocking motion
3. Remove adhesive residue from denture and mouth with warm water and a soft bristle brush
4. Use POLIDENT Denture Cleanser to thoroughly clean your denture and then rinse with water

- Poligrip seals out 74% of food particles\*<sup>1</sup>
- Poligrip gives users up to 38% increase in bite force\*\*<sup>2</sup>



## POLIDENT 3-MINUTE DAILY DENTURE CLEANSER

- STEP 01** Drop one Polident cleanser tablet into enough very warm (not hot) water to cover denture or partial
- STEP 02** After soaking for the time indicated, brush denture with solution using a soft brush. Discard remaining solution immediately after use
- STEP 03** Rinse denture thoroughly with running water

**#1 DENTIST  
RECOMMENDED  
BRAND OF DENTURE  
CLEANSER**



- Polident 3-Minute Daily Denture Cleanser kills 99.99% of odor-causing bacteria on dentures<sup>+3,4</sup>
- Dentures are 10x softer than natural teeth<sup>5</sup> and can be scratched by regular toothpaste



\*Vs. no adhesive after 1 hour on a lower full denture. \*\*Vs. no adhesive after 1 hour. †In laboratory tests.

References: 1. Haleon Data on File, 2011. 2. Munoz C et al. Prosthodont 2012; 21: 123-9. 3. Haleon Data on File, 2010. Micro Study, MD#056-10, Ignar & Ona. 4. Haleon Data on File, 2012. MD#012-2395. 5. Aleksandrova K, 2012. Literature review. 6. Charman KM et al. Letter in Applied Microbiology. 2009; 48:472-477.

Trademarks are owned by or licenced to the Haleon group of companies. PM-US-POLD-23-00008



To learn more, visit [mydenturecare.com](http://mydenturecare.com)