

Eroxon[®]

The First-and-Only ED Treatment Available Without a Prescription

- New Eroxon[®] treatment gel for erectile dysfunction (ED) is clinically proven to work within 10 minutes*.
- FDA cleared. Safe for you and your partner.
- Easy-to-apply gel in a single-dose tube.
- No prescription needed. Available in-store and online.

*For physical intimacy. Individual results may vary.



How to use Eroxon[®]



1

Dispense Gel

Unscrew the cap and pierce the tube seal with the top of the cap. Squeeze all contents onto your finger from the single-dose tube.



2

Massage

Gently massage Eroxon[®] gel directly onto the head of the penis for 15 seconds. Sexual stimulation is needed. Eroxon[®] works even better when used with your partner.



3

Play

Relax and have fun building intimacy with your partner. The fast-acting gel quickly evaporates and goes to work in about 10 minutes.



4

Connect

Enjoy the intimate moments with your partner. Be patient if Eroxon[®] doesn't help the first time. It may take using the product on a few separate occasions before you achieve the desired effect.



Help & FAQs

For more information and frequently asked questions, scan QR code or go to Eroxon.us

